



EDGEENDO®

EDGEBROACHES™

Product Type: Barbed Broaches

Specification: #0, #1, #2, #3, #4, #5, #6

Application: Use for endodontic treatment. This product is a medical product for dental treatment only, applicable for various pulpitis, inflammation etc. Remove and clean dental pulp cavity tissues.

Product structure and usage: Composed by file needle and file handle. Needle instrument for processing tissues inside root canal. Working length of the instrument have good durability and corrosion resistance, can satisfy the demand on removing and cleaning dental pulp cavity tissues for endodontic treatment. File handle can be sterilize by various liquor antiseptic or 145 degrees high temperature sterilizer.

Cautions:

1. This product can only use by doctor with formal endodontic treatment training and knowledge
2. Broaches should explore the root canal slowly with cautions, enforcement is strictly prohibited to avoid file breakage or stab into the root canal structure.
3. Product is not sterilized, new products need to be sterilized before use;
4. Sterilization method include autoclave or use antiseptic;
5. If apply autoclave method, heating temperature should not over 145 degrees or it will result in file handle melting and fortify parts depreciate
6. Sterilization should take place before every usage. Should take replacement if any instrument deformation is discovered.
7. After usage this product need to be treat as medical discard.

Transport and storage: Product transportation cannot damage product's original package.; store under indoor temperature; restricted for pressure

Manufacture date: See package

Period of Validity: 5 Years under recommended storage condition



EDGEENDO®
Twice as Good. Half the Cost.

1-855-985-3636

EDGEENDO.COM