



Paper points are an adjunct material for use in the drying of a prepared root canal or transferring dental materials into canal before obturation.

Hand-rolled
Highly absorbent
Variety sizes and packages available
Color coded

INDICATION AND PURPOSE

1. Paper points are used to dry the root canal before obturation.
2. Can be used for transferring sealer or cementation material to clean canal walls.

INSTRUCTION FOR USE

1. Measure the length of root canal
2. The formation of the root canal of the same size as the last file selected
3. Dry the root canal using paper points
4. About 10 ~ 15 points is used per root canal
5. Sealer or cement is applied by using paper points

CAUTIONS

1. Should use sterilized paper points
2. Do not use bent or broken points
3. Should be careful to avoid overfilling or into dead space when applied sealer or cement
4. Use immediately when applied sealer or cement
5. After using the paper points, unused should be kept in a cool place
6. Do not expose to direct sunlight

STORAGE

Store at room temperature of 1°C~30°C(33°F~86°F)

SHELF LIFE

4 years from manufacture date

Symbols

-  Use By
-  Batch Code
-  Sterilization using irradiation
-  Consult operating instructions
-  Do not reuse
-  Do not resterilize
-  Manufacturer
-  Date of manufacture
-  Keep away from sunlight
-  Temperature limitation

