

EDGE UTOPIA™

Enhanced Technology. Remarkably Beautiful. Ultra Secure Handle.

Our Greatest Endodontic Systems Ever!

Introducing Our New FireWire Blaze Blue Heat Treatment

EDGESEQUEL UTOPIA™

Glide
Path

Initial
Shaping

Optional
Shaping



15/04



25/04



30/04

or



25/06



EDGEENDO®

Order Now


Free Shipping

EdgeUtopia.com

EDGE UTOPIA™

The Remarkable EdgeSequel Utopia™

The New EdgeSequel Utopia™ delivers a Technology-Enhanced Constant .02, .04, .06 Taper System with a Remarkably-Beautiful blue blade featuring our FireWire Blaze Blue heat treatment. Plus, we added our new ultra secure, Ultra-Fit handle and shank.

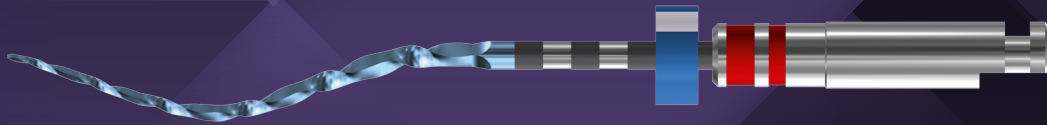


FireWire Blaze Blue

FireWire Blaze Blue is our newest proprietary heat treated NiTi. You will find that its flexibility, and feel provides unparalleled performance and shapes.

—It Is Truly, Amazing!

Pre-Sterilized

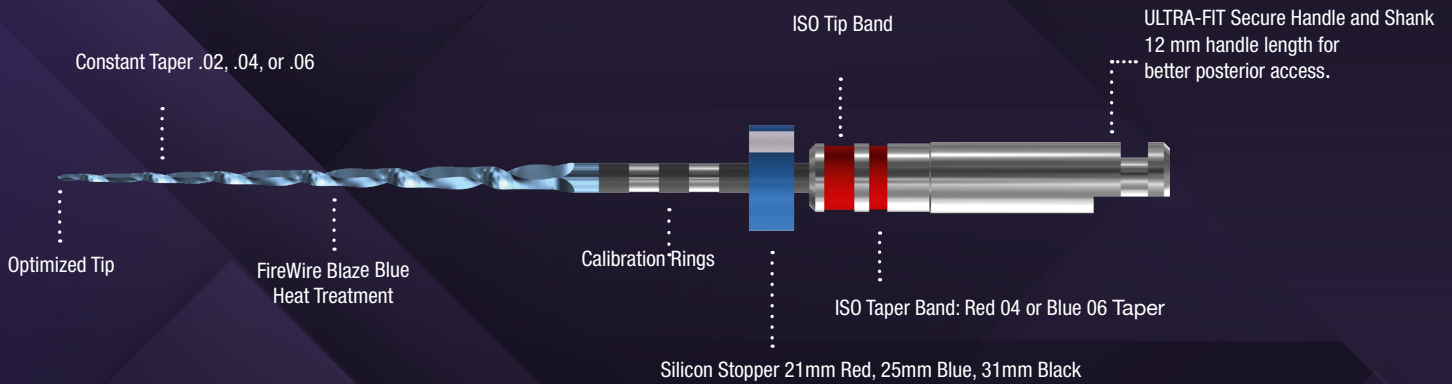

Triangular Cross Section

Available in 3 lengths: 21, 25 & 31 mm



Highly Flexible



No Bounce Back



High Strength



EDGEENDO®

Order Now

Free Shipping


EdgeUtopia.com

EDGESEQUEL UTOPIA™

Glide
Path

Initial
Shaping

Optional
Shaping

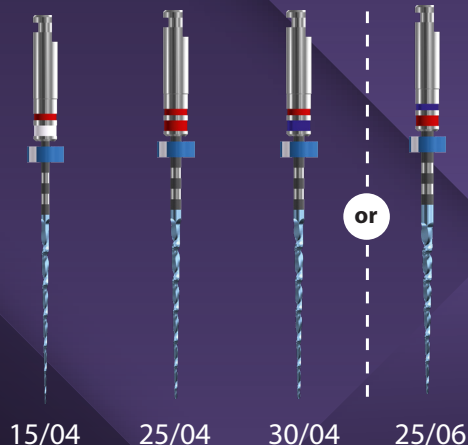

Clockwise rotation
Speed 800 to 1000 rpm
Torque setting 250 to 410 g/cm

Pre-Sterilized

Simplified Technique Guide

1. Create straight line access to canal orifices.
2. Locate canals and explore using stainless steel hand instruments. Minimum size #10K file to working length recommended prior to rotary file use.
3. Irrigate before each hand or rotary file.
4. Use #15/04 file in one or more passes, alternating with small-sized hand files if necessary, until working length is reached. If more coronal flare is desired it can be achieved by incorporating the EdgeGlidePath file used in a brushing motion.
5. Next use #25/04 to working length passively; if instrument has not reached working length use additional shaping instrument #20/04 to working length.
6. If #25/04 reached working length with minimal resistance or if clinician desires a larger apical shape additional instruments can be used (#30, #35, #40, etc).

*This technique is intended to be used as a guide only. Please see full IFU.

The New EdgeSequel Utopia™ is taking the original EdgeSequel Sapphire to the next level of performance, cutting ability and flexibility. We are also excited to introduce our incredible FireWire Blaze Blue NiTi along with our new ultra secure, Ultra-Fit handle and shank. This is our best EdgeSequel ever!

We know you will love the New EdgeSequel Utopia™!



EDGEENDO®

Order Now

Free Shipping

EdgeUtopia.com

EDGESEQUEL UTOPIA™

- Available in .02, .04 and .06 Constant Taper
 - Variable Pitch
- Triangular Cross Section
 - Maximizes file cutting efficiency
- Non-Cutting Tip
- Reduced handle and shank length for increased posterior access
- ISO tip size 15-50
- Available lengths: 21, 25 & 31 mm
- FireWire Blaze Blue technology
- Sterile Packaging
- Utopia alternative to the EdgeSequel Sapphire™
- Same technique as EdgeSequel Sapphire™
- ULTRA-FIT Secure Handle and Shank
- Spectacular FireWire Blue Blades
- Pre-Sterilized
- Free Shipping

6-Pack		STERILE		
		Length		
Tip Size	Taper Size	21 mm	25 mm	31 mm
15	04	ES150421SE	ES150425SE	ES150431SE
25	04	ES250421SE	ES250425SE	ES250431SE
	06	ES250621SE	ES250625SE	ES250631SE
30	04	ES300421SE	ES300425SE	ES300431SE
	06	ES300621SE	ES300625SE	ES300631SE
35	06	ES350621SE	ES350625SE	ES350631SE
40	04	ES400421SE	ES400425SE	ES400431SE
50	04	ES500421SE	ES500425SE	ES500431SE
Assorted Pack (15/04, 25/04, 30/04)		ASST15300421SE	ASST15300425SE	ASST15300431SE
Assorted Pack (15/04, 25/04, 25/06)		ASST15250621SE	ASST15250625SE	ASST15250631SE

Use Constant Taper, 04 or 06 Gutta Percha, Paper Points, or Obturators, or the same as EdgeSequel Sapphire™



EDGEENDO®

Order Now

Free Shipping

EdgeUtopia.com