

# EDGE UTOPIA<sup>®</sup>

Enhanced Technology. Remarkably Beautiful. Ultra Secure Handle.

## Our Greatest Endodontic Systems Ever!

Introducing Our New FireWire Blaze Heat Treatment

# EDGETAPER BLAZE UTOPIA<sup>™</sup>

SX

S1

S2

F1

F2



Pre-Sterilized

Alternative to ProTaper<sup>®</sup> Gold



EDGEENDO<sup>®</sup>

Order Now

Free Shipping

EdgeUtopia.com

ProTaper<sup>®</sup> Gold is a registered trademark of Dentsply Tulsa Dental Specialties

# EDGE UTOPIA<sup>®</sup>

## The Remarkable EdgeTaper Blaze Utopia™

The NEW EdgeTaper Blaze Utopia™ is an alternative to ProTaper Gold<sup>®</sup>, with more efficient, resilient blade, than EdgeTaper Platinum™. It can be used with the same technique as ProTaper Gold<sup>®</sup> and EdgeTaper Platinum™.


It features our FireWire Blaze Heat-Treatment making it efficient, safe, and flexible. Plus, we added our new ultra secure, Ultra-Fit Handle and all EdgeUtopia™ files are provided in pre-sterilized blister packs.



### FireWire Blaze

FireWire Blaze—Years to develop and worth every minute. We wanted to outdo our original FireWire and with FireWire Blaze, we believe we have! We know you will love the feel and flexibility as you negotiate the most challenging canals.

### Pre-Sterilized

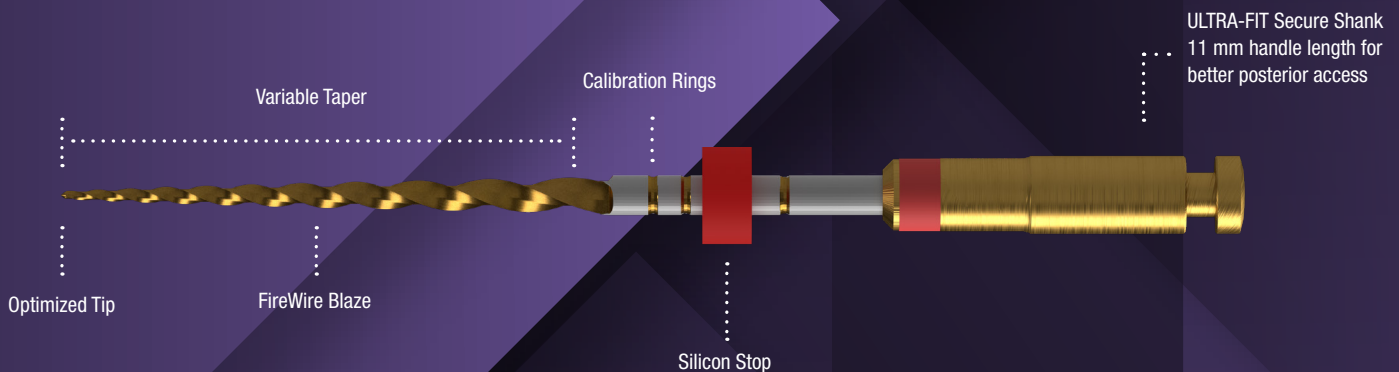

Tri-Shaped Cross Section

Available in 3 lengths: 21, 25 & 31 mm



Highly Flexible



No Bounce Back



High Strength



EDGEENDO<sup>®</sup>


Order Now

Free Shipping

EdgeUtopia.com

ProTaperGold<sup>®</sup> is a registered trademark of Dentsply Tulsa Dental Specialties

# EDGETAPER BLAZE UTOPIA™

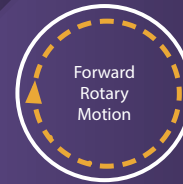

Speed 300 to 400 rpm  
Torque setting 250 to 510 g/cm

## Simplified Technique Guide

- Use EdgeTaper Blaze Utopia™ in canals with a confirmed, reproducible glide path.
- Use EdgeTaper Blaze Utopia™ in a deliberate brushing motion on the out-stroke.
- Irrigate, recapitulate with a #10 file and re-irrigate after removing each rotary file.
- Frequently clean and inspect the flutes of the rotary file.
- Use SX to eliminate triangles of dentin when present.

## Create straight line access to canal orifice.

1. NaOCl, 17% EDTA liquid or other viscous chelator should always be in the canal.
  2. Form a coronal glidepath using #10 & 15 hand instrument 2/3 the working length.
  3. Alternatively, mechanized glidepath files may be used instead.
  4. In the presence of viscous chelator, take the S1 the length taken by the #15.
  5. Then take the S2 the length taken by the #15.
  6. Established working length in the presence of a viscous chelator by confirming the working length with an apex locator and at least #10 hand instrument.
  7. Form a glidepath using #10 & 15 hand instrument to the working length.
  8. In the presence of viscous chelator, take the S1 to the working length.
  9. Then take the S2 to the working length.
  10. Use F1 Finishing until working length is reached.
  11. Gauge the foramen with a #20 hand instrument. If the instrument is snug at length, the canal is shaped and ready to be obturated.
- If the #20 hand instrument is loose at length, proceed to the F2 and, when necessary the F3.
  - The EdgeTaper Blaze Utopia™ sequence is always the same regardless of the length, diameter or curvature of the canal.

\*This technique is intended to be used as a guide only. Please see full IFU.

**The NEW EdgeTaper Blaze Utopia™ features metallurgically enhanced capabilities. We are also excited to introduce our incredible FireWire Blaze NiTi with our new FireWire Blaze heat treated alloy along with our new Ultra-Fit handle. The EdgeTaper Blaze Utopia™ has been designed to be improved over our EdgeTaper Platinum™!**

**We Know You Will Be Impressed!**

# EDGETAPER BLAZE



# UTOPIA™

- Alternative to ProTaper Gold® and EdgeTaper Platinum™
- Available sizes: SX, S1, S2, F1, F2, F3
- Triangle-shaped cross-section
- Improved cutting efficiency
- Resilient to unwinding
- Non-cutting tip
- Reduced shank and handle length for increased posterior access
- Available lengths: 21, 25 & 31 mm
- FireWire Blaze technology
- Sterile Packaging
- ULTRA-FIT Secure Handle
  - Allows for real time apex locator readings
- Spectacular Gold Color Heat Treatment
- Pre-Sterilized

6-Pack	STERILE	Tip Size	Taper Size	Length		
				21 mm	25 mm	31 mm
	SX	19	04v	19 mm Only ETBUSX19SU		
	S1	18	03v	ETBUS121SU	ETBUS125SU	ETBUS131SU
	S2	20	04v	ETBUS221SU	ETBUS225SU	ETBUS231SU
	F1	20	07v	ETBUF121SU	ETBUF125SU	ETBUF131SU
	F2	25	08v	ETBUF221SU	ETBUF225SU	ETBUF231SU
	F3	30	09v	ETBUF321SU	ETBUF325SU	ETBUF331SU
Assorted Packs				ETBUA21SU	ETBUA25SU	ETBUA31SU

**Check out our EdgeFlow™ & EdgePack™ Equipment to enhance your obturation needs.**

**Ask about our Compatible Gutta Percha and Paper Points!**



**EDGEENDO®**

**Order Now**

**Free Shipping**

**EdgeUtopia.com**