

EDGE UTOPIA[®]

Enhanced Technology. Remarkably Beautiful. Ultra Secure Handle.

Our Greatest Endodontic Systems Ever!

Introducing Our New FireWire Blaze Heat Treatment

EDGE X7 UTOPIA[™]

Glide
Path

Initial
Shaping

Optional
Shaping




17/04



25/04



30/04

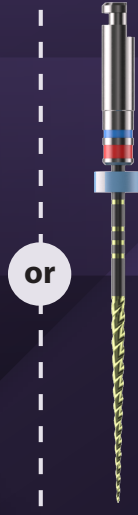

25/06

or



EDGEENDO[®]

Order Now

Free Shipping

EdgeUtopia.com

EDGE UTOPIA[®]

The Remarkable EdgeX7 Utopia™



The New EdgeX7 Utopia™ delivers a Technology-Enhanced EdgeFile X7[®] with a Remarkably-Beautiful gold blade featuring our FireWire Blaze heat treatment. Plus, we added our new ultra secure, Ultra-Fit handle and shank.



FireWire Blaze

FireWire Blaze—Years to develop and worth every minute. We wanted to outdo our original FireWire and with FireWire Blaze, we believe we have! We believe you will love the feel and flexibility as you negotiate the most challenging canals.

Pre-Sterilized

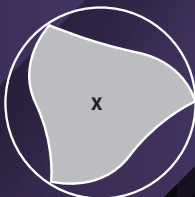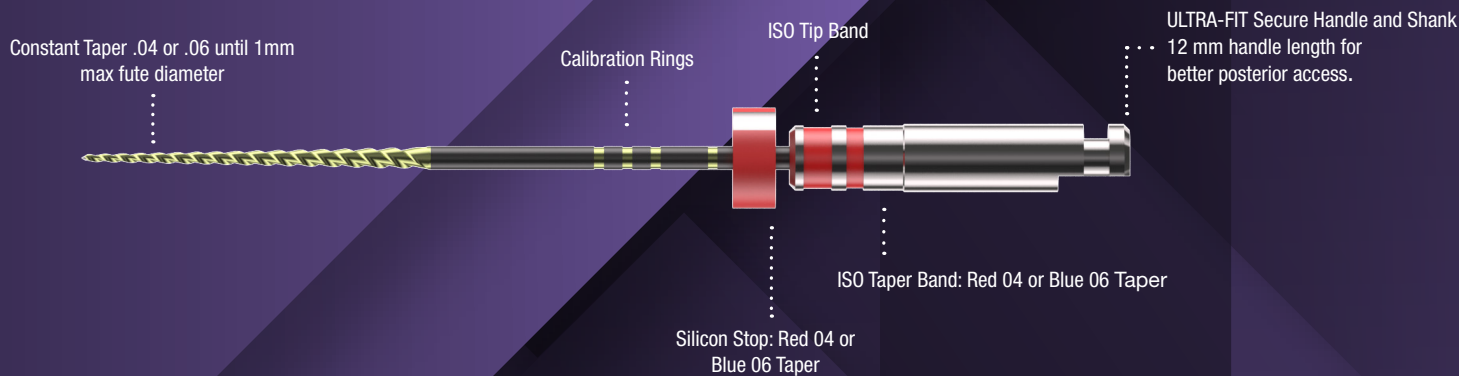

Parabolic Cross Section

Available in 3 lengths: 21, 25 & 29 mm, 04 & 06 File Size

Maximum flute diameter 1 mm



Highly Flexible



No Bounce Back



High Strength



EDGEENDO[®]

Order Now

Free Shipping

EdgeUtopia.com

EDGE X7 UTOPIA™

Glide
Path



17/04

Initial
Shaping



25/04

Optional
Shaping



30/04

or



25/06



Clockwise rotation
Speed 300 to 500 rpm
Torque setting 250 to 410 g/cm

Simplified Technique Guide

1. Create straight line access to canal orifices.
2. Locate canals and explore using stainless steel hand instruments. Minimum size #10K file to working length recommended prior to rotary file use.
3. Irrigate before each hand or rotary file.
4. Use #17/04 file in one or more passes, alternating with small-sized hand files if necessary, until working length is reached. If more coronal flare is desired it can be achieved by incorporating the EdgeGlidePath file used in a brushing motion.
5. Next use #25/04 to working length passively; if instrument has not reached working length use additional shaping instrument #20/04 to working length.
6. If #25/04 reached working length with minimal resistance or if clinician desires a larger apical shape additional instruments can be used (#30, #35, #40, etc).

*This technique is intended to be used as a guide only

The New EdgeX7 Utopia™ is taking the original X7 to the next level of performance, cutting ability and flexibility. We are also excited to introduce our incredible FireWire Blaze NiTi along with our new ultra secure, Ultra-Fit handle and shank. This is our best X7 ever!

We know you will love the New EdgeX7 Utopia™, and so will your patients!

EDGE X7 UTOPIA™

- Available in .04 and .06 Constant Taper until 1mm max diameter
 - Variable Pitch
- Maximum flute diameter 1mm allows for minimally invasive preparation
- Parabolic Cross Section non cutting tip
 - Maximizes file cutting efficiency
- Reduced handle and shank length for increased posterior access
- ISO tip size 17-45
- Available lengths: 21, 25 & 29 mm
- FireWire Blaze technology
- Sterile Packaging
- Replaces EdgeFile® X7
- Same technique as EdgeFile® X7
- ULTRA-FIT Secure Handle and Shank
- Spectacular color coating finish
- Pre-Sterilized
- Free Shipping

6-Pack	STERILE	Tip Size	Taper Size	Length				
				21 mm	25 mm	29 mm		
		17	04	X7U170421SE	X7U170425SE	X7U170429SE		
			06	X7U170621SE	X7U170625SE	X7U170629SE		
		20	04	X7U200421SE	X7U200425SE	X7U200429SE		
			06	X7U200621SE	X7U200625SE	X7U200629SE		
		25	04	X7U250421SE	X7U250425SE	X7U250429SE		
			06	X7U250621SE	X7U250625SE	X7U250629SE		
		30	04	X7U300421SE	X7U300425SE	X7U300429SE		
			06	X7U300621SE	X7U300625SE	X7U300629SE		
		35	04	X7U350421SE	X7U350425SE	X7U350429SE		
			06	X7U350621SE	X7U350625SE	X7U350629SE		
		40	04	X7U400421SE	X7U400425SE	X7U400429SE		
			06	X7U400621SE	X7U400625SE	X7U400629SE		
		45	04	X7U450421SE	X7U450425SE	X7U450429SE		
			06	X7U450621SE	X7U450625SE	X7U450629SE		
		Assorted Packs .04 (17, 20, 25, 25, 30, 35)				X7UASST0421SE	X7UASST0425SE	N/A
		Assorted Packs .06 (17, 20, 25, 25, 30, 35)				X7UASST0621SE	X7UASST0625SE	N/A

Check out our EdgeFlow™ & EdgePack™ Equipment to enhance your obturation needs.

Ask about our Compatible Gutta Percha and Paper Points!



EDGEENDO®

Order Now

Free Shipping

EdgeUtopia.com

Rev. E 04/2024